

Digital Display Loop

Information/Transaction Counter Loop System



Specific Benefits

- ✓ Simple, unobtrusive system- does not detract from interior design or the useable counter area
- ✓ Always "ON", no need for customer to request assistance
- ✓ Clearly defined area of induction loop transmission which maintains privacy of communication.
- ✓ Sophisticated electronic induction loop circuitry; the best possible hearing enhancement.
- ✓ Simple to fit because it can be installed easily throughout an organization with many locations.
- ✓ Allows for client confidentiality; better complying with HIPPA rules.
- ✓ Display feature expands benefits and uses.

contacta ■

332 East Lakewood Blvd., Suite 400
Holland, Michigan 49424

Phone (616) 392-3400
Fax (616) 392-3502
sales@contactainc.com

Description

Contacta Digital Display Loop system is designed to sit on the counter or desk, and can be hung on a wall. This system provides unobtrusive and continual assistance to hearing aid wearers. It is ideal for use in high traffic areas; for example, at reception counters or information desks, as well as in pharmacies and doctor's offices.

The system is designed to remain 'ON' continuously. Hearing aid wearers do not have to request assistance: once inside the field of the hearing loop, they are automatically included as soon as they switch their hearing aid to the 'T' position.

The Digital Display can be programmed to alternate between directions for use by the hearing aid wearer and promotional information. For example, upcoming events, advertising ads, further instructions, or answers to frequently asked questions.

The system is configured for one-to-one conversation involving a counter/service person and the hearing aid wearer. The positioning of the sign controls the extent and direction of the field, and it can be used at single position counters as well as multiple person counters. For multiple position counters, each position has its own induction loop, so privacy is maintained between counters.

The system includes the Digital Display, amplifier, interconnected cables, microphone, and installation accessories (not including tools).



All information is believed to be correct at the time of assembly; however, Contacta, Inc. reserves the right to change any specification as part of our program of continuous improvement.