



# MY - T

## LISTENING SYSTEM

The enjoyment of watching shows and recordings on television can be seriously affected if you cannot hear the sound properly because of hearing impairment - even if you are wearing a hearing aid. Sound can be muffled and distorted and often the volume of the television is increased in order to try and resolve the problem. This can often be uncomfortable for others who are also watching but do not have a hearing impairment. The My-T listening system is aimed at providing first class sound reception from the television or other audio system for people who wear hearing aids fitted with a telecoil (T) or tephone coil..

The system consists of two main components: the transmitter, which connects to the television or audio system, and the receiver, which is placed in close proximity to the hearing aid wearer. The system is very simple to set up and does not require complex installation to the room or trailing cords on the floor.

The transmitter is connected to the television or audio system via RCA jacks. It then picks up the sound and sends it to the receiver wirelessly using infra red technology. The receiver is presented in an attractive portable housing which can be placed on a table, chair or even the floor providing the unit is within a distance of approximately four feet from the user. Sound is transmitted from the receiver as a magnetic field which connects to the telecoil in the hearing aid. The listener can adjust the volume to the required level via the hearing aid without increasing the sound level of the television itself.

With a My-T, listening to the television or audio system is clear and comfortable for everybody in the room!

## COMFORT THROUGH CLARITY